


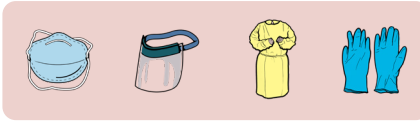

## N95, face shield, contact gown, gloves

# Checklist

- ① **Remove** any personal items and jewelry and put in secure location, not in pockets.
- ② If you already have a mask on: **Sanitize** hands. **Remove** mask. **Store** mask. **Sanitize** hands.
- ③ **Put on** contact gown outside room. Open-end **faces** your back. **Tie** the back of the gown.
- ④ **Put on** gloves over the cuffs of the gown.
- ⑤ **Put on** N95, ensuring proper seal. **Ensure** straps are not crossing.
- ⑥ **Place** hands over the front of the N95. **Breathe** an easy deep breath in and out. If you **feel** air escape the edges, **refit** and **repeat**.
- ⑦ **Sanitize** gloves.
- ⑧ **Put on** face shield.
- ⑨ **Sanitize** gloves.
- ⑩ **ENTER room. DO NOT enter** the room if you do not achieve a proper respirator seal.
- ⑪ **Do not** touch face or re-adjust N95 or face shield inside room.



**N95, face shield, contact gown, gloves**

## Checklist

### Inside room

- 1 **Sanitize** gloves.
- 2 Cross arms and **grip** gown on shoulders. **Pull** and **break** gown in controlled fashion.
- 3 **Roll** the gown towards your hands. **Remove** the gloves with the gown. **Dispose** of gloves and gown.
- 4 **Sanitize** hands.
- 5 **Exit** patient room.

### Outside room

- 1 **Put on** new gloves.
- 2 **Place** wipe on table.
- 3 **Remove** eye wear and **place** on wipe. **Do not** touch face.
- 4 **Sanitize** gloves.
- 5 **Pinch** bottom strap and **pull** far over head. **Do not** let straps touch your face.
- 6 **Pinch** top strap and **pull** far over head. **Do not** let straps touch your face as you **remove** the N95.
- 7 **Store** N95.
- 8 **Sanitize** gloves.
- 9 **Put on** mask.
- 10 **Wipe** shield. **Sanitize** in the following order: back of shield, elastic band, foam band, front of shield.
- 11 **Place** shield in bag to dry.
- 12 **Wipe** table.
- 13 **Sanitize** gloves.

- 14 **Remove** gloves.
- 15 **Wash hands** with soap and water or **sanitize** hands.